



## TECHNICAL SERVICES BULLETIN

020-004 MT204 Water Level Inspection  
March 21, 2002

---

2711 Citrus Road , Rancho Cordova, CA 95742 (916) 858-5100

---

**Scope:** All MT204 Plasma Thawers.

**Subject:** *MT204 Plasma Thawer Water Level Check*

The fluid level indicator in the MT204 is a clear rod attached to the bottom of the sleeve inside the fill port. Water is poured into the reservoir through the fill port until the water level reaches the bottom of the rod.

After the initial fill, the water level should be inspected immediately after a drain cycle. This will give the proper indication of the level. Do not add water unless the level has been checked immediately after the drain cycle.

The MT204 has a low fluid indicator that will notify the operator if the water level drops below an acceptable level.

### **Procedure**

#### **1. Monthly Preventive Maintenance – Check Water Level**

- Check the water level immediately after a drain cycle is completed. Add water if needed.

#### **2. Quarterly Preventive Maintenance**

- Follow the Operators Manual to change the water in the reservoir. Fill with water to the bottom of the rod.

Please note this information in your Operator Manual.

If you have any questions, contact THERMOGENESIS CORP. Customer Service at 800-783-8357 (U.S. and Canada) or 916-858-5100 (non-U.S./Canada) for assistance.