

**Scope:**

This bulletin covers the CryoSeal® FS System CP-3 Plasma Processing Disposable (part # 8-0360-0).

**Background:**

It may be possible to damage the outer protective pouch of the frozen fibrin sealant syringe sets during storage, transportation or handling. Care must be taken.

To prevent compromising the sterility of the fibrin sealant product, please follow the frozen syringe set proper handling procedure below.

**Procedure:**

Follow the CP-3 Processing Disposable instructions with **these additional steps:**

1. When removing frozen product from the freezer, ensure that the pouch is protected so that sharp or heavy objects do not puncture it.
2. If transporting the frozen syringe set:
  - Wrap it in bubble wrap
  - Place a layer of protective material between the syringe set and the dry ice. The dry ice should always be on the bottom.

**Contact Information:**

If you have any questions, contact THERMOGENESIS CORP. Technical Service at 800-783-8357 (U.S. and Canada) or 916-858-5100 (non-U.S./Canada), or fax to 916-858-5199 for assistance.