



TECHNICAL SERVICES BULLETIN

020-005 Thawer Disinfectant Change
November 1, 2002

2711 Citrus Road , Rancho Cordova, CA 95742 (916) 858-5100

Scope: All THERMOGENESIS CORP. MT202, MT204, and MT210 Plasma Thawers.

Subject: *Disinfectant Change*

Currently, disinfectant tablets or liquid are used for the THERMOGENESIS CORP. Plasma Thawers to reduce bacterial growth between maintenance periods.

THERMOGENESIS CORP. has identified a new liquid disinfectant to be used in all thawer models. Therefore, we are discontinuing the disinfectant tablets and the one-gallon liquid disinfectant previously supplied. The new liquid disinfectant is more effective against bacterial growth and easier to use. The new liquid comes in a 4-oz. bottle and can be ordered from THERMOGENESIS CORP. using part number 5-34-002.

The following table shows the amount of the new liquid disinfectant to use for each thawer model:

Plasma Thawer Model (110V and 220V)	Amount of New Liquid Disinfectant per Unit	Water Amount per Unit
MT202	2 ml	3 gallons
MT204	5 ml	8 gallons
MT210	20 ml	30 gallons

Procedure

Upon installation, at the next Quarterly Maintenance, or at the time of replacement of the water, use the new liquid disinfectant according to the concentration amounts listed in the table above. It is no longer necessary to test the concentration level of the disinfectant in your thawer as part of Monthly Maintenance.

Please update this information in your Operator Manual.

If you have any questions, contact THERMOGENESIS CORP. Customer Service at 800-783-8357 (U.S. and Canada) or 916-858-5100 (non-U.S./Canada) for assistance.